LEJON KEMI

Cleanox surface disinfectant

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Lejon Kemi AB – Development and production of environmentally adapted chem.-technical products

Fritz Janssons väg 20, SE-184 70 Åkersberga, Sweden – Customer contact ph. no. + 46 (0)76-827 00 96

E-mail: info@lejonkemi.se Home page: www.lejonkemi.se

LEJON KEMI **Cleanox** surface disinfectant

(ENG) Cleanox is an effective disinfectant against a wide range of both gram negative and gram positive bacteria, fungi (mold) and many types of viruses. Cleanox is specially developed for the disinfection of breathing masks. A working dilution of Cleanox and FPG Wash can also be used for manual disinfection of most materials that tolerate water, such as plastics, rubber, stainless steel, aluminium, glass, ceramics, stones, lacquered and painted surfaces, as well as most types of textiles. The agent was tested according to European Standards EN 1040, EN 1276, EN 14476, EN 13697, EN 1656, EN 1657, EN WI 15 and Fugicidal activity.

Disinfection in washing machines: Install the Lejon Kemi washing program for breathing masks with disinfection steps and dosage pump for Cleanox. Use Lejon Kemi's liquid detergent FPG Wash for washing in washing machines. When machine washed, breathing masks must be placed in Lejon Kemi's washing bags, specially developed for the washing and disinfection of breathing masks.

Manual disinfection: Prepare a working dilution according to the dosing instructions below with FPG Wash and Cleanox in hot water 35–45 °C. Stir thoroughly. Soak the breathing mask/item in the dilution for about 15 minutes. Work over the surface with a soft brush or sponge and rinse carefully in clean water. Hot water increases the biocide effect. **Note! Wear protective gloves of plastic or rubber.**

Dosage: Disinfection in washing machines: 0.4% Cleanox. (Use the washing program for breathing masks with disinfection steps).

Manual disinfection: 1% FPG Wash and 0.5% Cleanox. (50 ml FPG Wash + 25 ml Cleanox for 5 litres hot water (approx. 35–45°C)). Stir.

Drying: Thorough drying of breathing masks at 55°–60°C for at least 30 minutes limits the growth of many types of microorganisms. **Do not wash breathing masks with built-in electronics in washing machines. Cleanox should not be used on elastic bands as some types may lose their elasticity, especially after repeated disinfection with the agent.**

Environmental information: The product is rapidly biodegradable. The packaging can be recycled by sorting at source as hard plastic packaging.

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(ENG) Causes severe eye irritation

Wear eye/face protection IN THE EVENT OF CONTACT WITH EYES: Carefully flush with water for several minutes. Remove contact lenses if this can be done without difficulty. Continue flushing. If eye irritation persists, contact a physician. IN THE EVENT OF SKIN CONTACT: Wash with plenty of water. Remove contaminated clothing. **Contains:** e-phthalimido-peroxy-hexanoic acid 15–30%, dinatrium salt of 1-Hydroxyethylidene-1,1-diphosphonic acid < 5%.

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Store tightly closed in the original container.

Protect from frost and direct sunlight.

Lejon Kemi AB – Development and production of environmentally adapted chem.-technical products

Fritz Janssons väg 20, SE-184 70 Åkersberga, Sweden – Customer contact ph. no. + 46 (0)76-827 00 96

E-mail: info@lejonkemi.se Home page: www.lejonkemi.se